



Mini Anchor™

GII Anchor™

SuperAnchor™

*For High Strength
Soft Tissue Fixation*

Mitek®
PRODUCTS

SHOULDER



Rotator Cuff Repair

KNEE



Patellar Ligament Repair

ANKLE



Syndesmosis Repair

SuperAnchor™

The Mitek SuperAnchor is indicated where exceptionally high fixation strength is required. The SuperAnchor uses four nitinol arcs for maximum strength while maintaining a small size.


SuperAnchors offer significant benefits for soft tissue repair:

- ♦ **High fixation strength.** Greater than the double strand knot pullout strength of #5 braided polyester suture. Mitek Anchors have been safe and effective through years of clinical experience.
- ♦ **Self-centering.** Allows equal distribution of load across all four arcs.
- ♦ **Suture protection.** Suture attachment is at the back of the Anchor, protecting the suture from bony abrasion.
- ♦ **Sliding suture.** Suture passes smoothly through the Anchor, simplifying assembly. Sliding knots can be used.
- ♦ **Press fit.** Subcortical purchase. No threads to strip.
- ♦ **Small size.** Allows placement of multiple suture fixation points. Significantly smaller than competitive designs.




- Actual size
- 2.9 mm x 18 mm drill hole ●
- Shown on cover with #5 suture
- Fixation strength (approx.) 42 lbs.

SuperAnchor vs #5 Suture

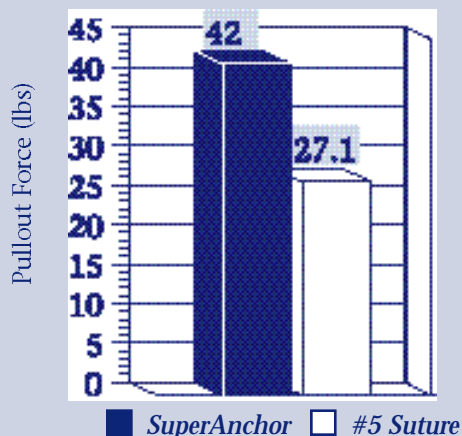


The Mitek SuperAnchor pullout strength versus the average knot-pull strength of #5 braided polyester suture (USP, two strands).


Mitek SuperAnchor Insertion

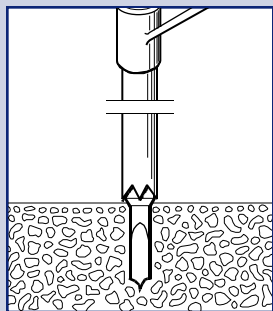


Drill hole.

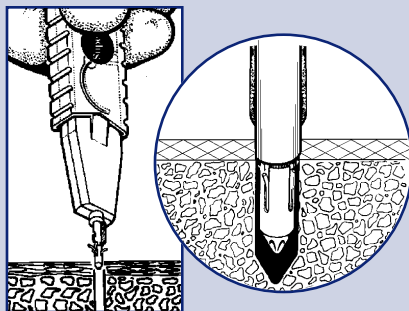

Insert Anchor.

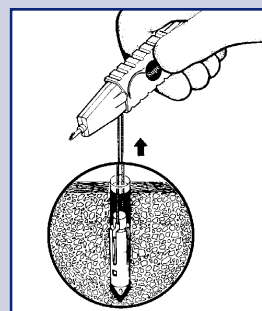

Pull suture from inserter.

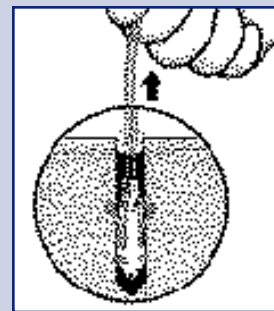

Set Anchor.

Ordering Information

Cat. No.	Description	Quantity
210409	SuperAnchor w/ threader tab	1/box
210309	SuperAnchor w/ threader tab	5/box
212209	Super QuickAnchor®	1/box
212309	Super QuickAnchor®	5/box
211113	SuperAnchor Drill (2.9 mm x 18 mm)	1/box
213107	GII/Super Drill Guide	1/box
212114	SuperAnchor Inserter	1/box
218113	Super Instrument Set Includes 2 Drills, 2 Inserters, 1 Drill Guide and 1 Tray	1/box

Extended Family – Extended Indications

Mitek®
PRODUCTS

ETHICON, INC.
a Johnson & Johnson company

Customer Service – (800) 382-4682

60 Glacier Drive, Westwood, MA 02090 USA
800 356-4835 / 617 461-9700 / Fax 617 461-9166